



**User instructions:**

- Remove the pre-assembled device from the box.
- Insert knob-1 into the device as shown in the image.
- Attach the 20d lens onto the movable lens ring.
- Insert the smartphone into the smartphone holder.
- Align the camera and flash of the smartphone in line with the 20d lens.
- Secure the smartphone using knob-2.
- Open the camera app and switch to video mode.
- Turn the flash on and align the lens with the patients eye using the eyebrow rest till the red reflex is observed on the phone.
- Move the lens ring towards the smartphone till you get desired view of the fundus.



**C3 Prototypes| Colpen Products Pvt. Ltd.**

421 Devpath Complex, Behind Lal Bungalow,  
Off CG Road, Ellisbridge, Ahmedabad 380006  
Gujarat, India

Phone: +91-79-26421191

Mobile: +91-9913933344

E-mail: [yash@c3prototypes.com](mailto:yash@c3prototypes.com)

Web: [www.c3prototypes.com](http://www.c3prototypes.com)



**FUNDUSCAM™**

PORTABLE FUNDUS CAMERA